

PLEASE READ THESE SPECIAL INSTRUCTIONS BEFORE FABRICATING THE ORTHOSES

THANK YOU FOR PURCHASING THE
BECKER ORTHOPEDIC
MODEL 1014 RATCHET LOCK™ OR
1015 KEY LOCK™ KNEE JOINT

ADDITIONAL FABRICATION CONSIDERATIONS

- **FOR THE JOINTS TO FUNCTION AS INTENDED, PLEASE
TAKE EXTRA TIME AND CAREFULLY SQUARE THE JOINTS
TO EACH OTHER IN ALL 3 PLANES .**
- **DO NOT PUT FLEXION THE PROXIMAL BAR.**
Flexing the proximal bar could elongate the rivet holes and foul
the locking mechanism.
- **ONLY REMOVE THE PIVOT SCREW.**
Trigger screws are bonded in place and may break if removal
is attempted.
- **SEAL THE JOINT WHILE FINISHING.**
Debris can get into the joint and foul the locking mechanism.



EC REP

Acorn Regulatory Consultancy
Services Limited
Knockmorris Cahir Co. Tipperary Ireland,
Postcode: E21 R766
P 012 4626 8456
F 012 4626 8648

