



**BECKER**  
A Tradition of Excellence. A Commitment to Innovation.

635 Executive Dr | Troy, MI 48083  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f t in

EC REP  
Acorn Regulatory Consultancy Services Limited  
Knoxtown Castle Cty, Eggerary  
Inland, Postcode: G21 8Y66  
P 012 4626 8436  
F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/2020



**"A Tradition of Excellence"**

Becker Orthopedic offers the largest selection of orthoses to enable you to precisely fit the needs of your patient.

This insert is perforated, so that the instructions may be given to your patient. When you deliver the completed orthosis, please discuss basic maintenance procedures with your patient and give him/her the section entitled "Patient Instructions."


Please write in product model and lot number information below. This section should be signed by the patient as proof he/she has been given maintenance instructions and then attached to your patient file.

Model #: \_\_\_\_\_ Lot #: \_\_\_\_\_

Patient Signature: \_\_\_\_\_

Date: \_\_\_\_\_

(Attach to Patient File at Delivery)




**BECKER**  
A Tradition of Excellence. A Commitment to Innovation.

635 Executive Dr | Troy, MI 48083  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f t in

EC REP  
Acorn Regulatory Consultancy Services Limited  
Knoxtown Castle Cty, Eggerary  
Inland, Postcode: G21 8Y66  
P 012 4626 8436  
F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/2020



**"Tradicija izvrsnosti"**

Becker Orthopedic nudi najveći izbor ortoza koji vam omogućuje da se precizno prilagodite potrebama svog pacijenta.

Ovaj umetak je perforiran, tako da se pacijentu mogu dati upute. Kad pružite kompletnu ortozu, razgovarajte sa svojim pacijentom o osnovnim postupcima održavanja i pokažite mu/joj odjeljak pod naslovom "Upute za pacijenta".

Molimo dolje upišite podatke o modelu proizvoda i broju serije. Ovaj odjeljak pacijent mora potpisati kao dokaz da je dobio upute za održavanje, a zatim ga priložiti u karton pacijenta.

Model# \_\_\_\_\_ Serija #: \_\_\_\_\_

Potpis pacijenta: \_\_\_\_\_

Datum: \_\_\_\_\_

(Priložite u karton pacijenata pri isporuci)



**BECKER**  
A Tradition of Excellence. A Commitment to Innovation.

635 Executive Dr | Troy, MI 48083  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f t in

EC REP  
Acorn Regulatory Consultancy Services Limited  
Knoxtown Castle Cty, Eggerary  
Inland, Postcode: G21 8Y66  
P 012 4626 8436  
F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/2020



**"Tradicija izvrsnosti"**

Becker ortopedija nudi najveći izbor ortoza koje vam omogućavaju da se precizno prilagodite potrebama svog pacijenta.

Ovaj umetak je perforiran, tako da se uputstva mogu dati vašem pacijentu. Kada isporučujete kompletnu ortozu, razgovarajte sa svojim pacijentom o osnovnim postupcima održavanja i dajte mu/joj odeljak pod naslovom „Uputstva za pacijenta“.

Molimo vas da ispod napišete podatke o modelu proizvoda i broju serije. Pacijent treba da potpiše ovaj odeljak kao dokaz da je dobio uputstva za održavanje, a zatim ga treba priložiti vašem kartonu pacijenta.


Model#: \_\_\_\_\_ Serija

#: \_\_\_\_\_

Potpis pacijenta: \_\_\_\_\_

Datum: \_\_\_\_\_

(Priložiti kartonu pacijenta pri isporuci)




**BECKER**  
A Tradition of Excellence. A Commitment to Innovation.

635 Executive Dr | Troy, MI 48083  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f t in

EC REP  
Acorn Regulatory Consultancy Services Limited  
Knoxtown Castle Cty, Eggerary  
Inland, Postcode: G21 8Y66  
P 012 4626 8436  
F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/2020



**"Tradicija izvrsnosti"**

Becker Orthopedic nudi najveći izbor ortoza koji vam omogućuje da se precizno prilagodite potrebama svog pacijenta

Ovaj umetak je perforiran, tako da se pacijentu mogu dati upute. Kad pružite kompletnu ortozu, razgovarajte sa svojim pacijentom o osnovnim postupcima održavanja i pokažite mu/joj odjeljak pod naslovom "Upute za pacijenta".

Molimo da u nastavku upišite podatke o modelu proizvoda i broju serije. Ovaj odjeljak pacijent mora potpisati kao dokaz da je dobio upute za održavanje, a zatim ga priložiti u karton pacijenta.

Model# \_\_\_\_\_ Serija #: \_\_\_\_\_

Potpis pacijenta: \_\_\_\_\_

Datum: \_\_\_\_\_

(Priložite u karton pacijenata pri isporuci)



635 Executive Dr | Troy, MI 48063  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f | in

EC REP  
Aspen Regulatory Consultancy  
Services Limited  
Kilcockmore Cahrl Co., Tipperary  
Ireland, Postcode: E21 R796  
P: 012 4528 8408  
F: 012 4528 8548

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/30/20



**Patient Instructions**  
(Give to Patient)

It is important to keep your joints clean (wipe with a dry, lint free cloth) to assure smooth motion. To clean, wash with a mild detergent, rinse and dry thoroughly. The joints should also be lubricated periodically with a synthetic lubricant. When not in use, your orthosis should be stored away from open flame or direct sunlight. Metal parts can conduct heat and plastic can deform or deteriorate prematurely due to heat effects. As a daily donning routine, you should perform the following safety check:

**Safety Check**

1. Inspect uprights for dents, scratches, or cracks.
2. Straps should be secure and have no tears.
3. Make sure all rivets and screws are tight.
4. Check the locks to assure engagement.

If you have any problems, notice any undue wear, or observe any unsafe or unusual condition in the orthosis, contact your Certified Orthotist immediately.

Orthotist: \_\_\_\_\_

Phone: ( ) \_\_\_\_\_

*Metal Ankle/Hip Joint*



635 Executive Dr | Troy, MI 48063  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f | in

EC REP  
Aspen Regulatory Consultancy  
Services Limited  
Kilcockmore Cahrl Co., Tipperary  
Ireland, Postcode: E21 R796  
P: 012 4528 8408  
F: 012 4528 8548

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/30/20



**Upute za pacijente**  
(Dati pacijentu)

Važno je da zglobovi budu čisti (obrišite ih suhom krpom koja ne ostavlja vlakna) kako biste osigurali glatko kretanje. Za čišćenje operite blagim deterdžentom, isperite i temeljito osušite. Spojeve treba povremeno podmazivati sintetičkim mazivom. Kada se ne koristi, ortozi treba čuvati dalje od otvorenog plamena ili izravne sunčeve svjetlosti. Metalni dijelovi mogu provoditi toplinu, a plastika se može deformirati ili prerano pokvariti zbog utjecaja topline. Kao svakodnevnu rutinu oblačenja trebali biste izvršiti sljedeće sigurnosne provjere:

**Sigurnosna provjera**


1. Pregledajte uporišta i zglobove koljena radi ulegnuća, ogrebotina ili pukotina.
2. Trake moraju biti sigurne i bez pucanja.
3. Provjerite jesu li sve zakovice i vijci dobro zategnuti.
4. Provjerite brave kako biste osigurali stabilnost.

Ako imate bilo kakvih problema, primijetite nepotrebno trošenje ili uočite bilo koje nesigurno ili neuobičajeno stanje u ortozi, **odmah** se obratite svom ovlaštenom ortopedu.

Ortoped: \_\_\_\_\_

Telefon: ( ) \_\_\_\_\_


Metalni zglob gležnja/kuka



635 Executive Dr | Troy, MI 48063  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f | in

EC REP  
Aspen Regulatory Consultancy  
Services Limited  
Kilcockmore Cahrl Co., Tipperary  
Ireland, Postcode: E21 R796  
P: 012 4528 8408  
F: 012 4528 8548

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/30/20



**Uputstva za pacijenta**  
(Dati pacijentu)

Važno je da zglobovi budu čisti (obrišite ih suvom krpom koja ne ostavlja dlačice) kako biste osigurali glatko kretanje. Za čišćenje operite blagim deterdžentom, isperite i dobro osušite. Zglobove treba povremeno podmazivati sintetičkim mazivom. Kada se ne koristi, ortozi treba čuvati dalje od otvorenog plamena ili direktne sunčeve svjetlosti. Metalni delovi mogu provoditi toplotu, a plastika se može deformisati ili prerano pokvariti zbog uticaja toplote. Kao svakodnevnu rutinu oblačenja, trebalo bi da izvršite sledeću bezbednosnu proveru:

**Bezbedonosna provera**

1. Proverite da li na uspravnim delovima ima udubljenja, ogrebotina ili pukotina.
2. Trake treba da su pričvršćene i da nisu poderane
3. Pobrinite se da su sve zakovice i vijci pričvršćeni.
4. Proverite brave da biste osigurali rad.

Ako imate bilo kakvih problema, primetite habanje pre vremena ili uočite bilo koje nesigurno ili neuobičajeno stanje u ortozi, **odmah** se obratite svom ovlašćenom ortopedu.

Ortoped: \_\_\_\_\_

Telefon: ( ) \_\_\_\_\_

Metalni zglob za članak/kuk



635 Executive Dr | Troy, MI 48063  
P: 800-521-2192 | 248-588-7480  
BeckerOrthopedic.com | f | in

EC REP  
Aspen Regulatory Consultancy  
Services Limited  
Kilcockmore Cahrl Co., Tipperary  
Ireland, Postcode: E21 R796  
P: 012 4528 8408  
F: 012 4528 8548

©2020 Becker Orthopedic Appliance Co.,  
All rights reserved.  
Revision 04/30/20



**Upute za pacijente**  
(Dati pacijentu)

Važno je da zglobovi budu čisti (obrišite ih suhom krpom koja ne ostavlja vlakna) kako biste osigurali glatko kretanje. Za čišćenje operite blagim deterdžentom, isperite i temeljito osušite. Zglobove treba povremeno podmazivati sintetičkim mazivom. Kada se ne koristi, ortozi treba čuvati dalje od otvorenog plamena ili izravne sunčeve svjetlosti. Metalni dijelovi mogu provoditi toplinu, a plastika se može deformisati ili prerano pokvariti zbog utjecaja topline. Kao svakodnevnu rutinu oblačenja trebali biste izvršiti sljedeće sigurnosne provjere:

**Sigurnosna provjera**

1. Pregledajte stubove i zglobove koljena radi ulegnuća, ogrebotina ili pukotina.
2. Trake moraju biti sigurne i bez pucanja.
3. Provjerite jesu li sve zakovice i vijci dobro zategnuti.
4. Provjerite brave kako biste osigurali stabilnost.

Ako imate bilo kakvih problema, primijetite nepotrebno habanje ili uočite bilo koje nesigurno ili neuobičajeno stanje u ortozi, **odmah** se obratite svom ovlaštenom ortopedu.

Ortoped: \_\_\_\_\_

Telefon: ( ) \_\_\_\_\_

Metalni zglob gležnja/kuka