635 Executive Dr | Troy, MI 48083  
 P: 800-521-2192 | 248-588-7480  
 BeckerOrthopedic.com | f t in

EC REP  
 Acorn Regulatory Consultancy Services Limited  
 Knockmore Carr Co, Tipperary  
 Ireland, Postcode: E21 R796  
 P 012 4626 8458  
 F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
 All rights reserved.  
 Revision 04/2020

**"A Tradition of Excellence"**

Becker Orthopedic offers the largest selection of orthoses to enable you to precisely fit the needs of your patient.

This insert is perforated, so that the instructions may be given to your patient. When you deliver the completed orthosis, please discuss basic maintenance procedures with your patient and give him/her the section entitled "Patient Instructions."

Please write in product model and lot number information below. This section should be signed by the patient as proof he/she has been given maintenance instructions and then attached to your patient file.

Model #: \_\_\_\_\_ Lot #: \_\_\_\_\_

Patient Signature: \_\_\_\_\_

Date: \_\_\_\_\_

*(Attach to Patient File at Delivery)*

635 Executive Dr | Troy, MI 48083  
 P: 800-521-2192 | 248-588-7480  
 BeckerOrthopedic.com | f t in

EC REP  
 Acorn Regulatory Consultancy Services Limited  
 Knockmore Carr Co, Tipperary  
 Ireland, Postcode: E21 R796  
 P 012 4626 8458  
 F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
 All rights reserved.  
 Revision 04/2020

**Patient Instructions**  
 (Give to Patient)

It is important to keep your joints clean (wipe with a dry, lint free cloth) to assure smooth motion. To clean, wash with a mild detergent, rinse and dry thoroughly. The joints should also be lubricated periodically with a synthetic lubricant. When not in use, your orthosis should be stored away from open flame or direct sunlight. Metal parts can conduct heat and plastic can deform or deteriorate prematurely due to heat effects. As a daily donning routine, you should perform the following safety check:

**Safety Check**

1. Inspect uprights for dents, scratches, or cracks.
2. Straps should be secure and have no tears.
3. Make sure all rivets and screws are tight.
4. Check the locks to assure engagement.

If you have any problems, notice any undue wear, or observe any unsafe or unusual condition in the orthosis, contact your Certified Orthotist immediately.

Orthotist: \_\_\_\_\_

Phone: ( ) \_\_\_\_\_

*Metal Ankle/Hip Joint*

635 Executive Dr | Troy, MI 48083  
 P: 800-521-2192 | 248-588-7480  
 BeckerOrthopedic.com | f t in

EC REP  
 Acorn Regulatory Consultancy Services Limited  
 Knockmore Carr Co, Tipperary  
 Ireland, Postcode: E21 R796  
 P 012 4626 8458  
 F 012 4626 8648

©2020 Becker Orthopedic Appliance Co.,  
 All rights reserved.  
 Revision 04/2020

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