BECKER ORTHOPEDIC

635 Executive Drive, Troy, MI 48083-4576 USA
(248) 588-7480 • Fax (248) 588-2960
www.BeckerOrthopedic.com

**ACORN REGULATORY CONSULTANCY
SERVICES LIMITED**

Knockmoris Cahir Co. Tipperary • Ireland • Postcode: E21 R766
Phone: 353 52 617 6706 • 353 52 614 8806

"A Tradition of Excellence"

Becker Orthopedic offers the largest selection of orthoses to enable you to precisely fit the needs of your patient.

This insert is perforated, so that the instructions may be given to your patient.

When you deliver the completed orthosis, please discuss basic maintenance procedures with your patient and give him/her the section entitled "Patient Instructions".

Please write in product model and lot number information below. This section should be signed by the patient as proof he/she has been given maintenance instructions and then attached to your patient file.

Model # _____ Lot # _____

Patient Signature: _____

Date: _____

(Please Attach to Patient File)

BECKER ORTHOPEDIC

635 Executive Drive, Troy, MI 48083-4576 USA
(248) 588-7480 • Fax (248) 588-2960
www.BeckerOrthopedic.com

**ACORN REGULATORY CONSULTANCY
SERVICES LIMITED**

Knockmoris Cahir Co. Tipperary • Ireland • Postcode: E21 R766
Phone: 353 52 617 6706 • 353 52 614 8806

Patient Instructions

(Give to Patient)

It is important to keep your joints clean (wipe with a dry, lint free cloth) to assure smooth motion. To clean, wash with a mild detergent, rinse and dry thoroughly. The Joints should also be lubricated periodically with a synthetic lubricant. When not in use, your orthosis should be stored away from open flame or direct sunlight. Metal parts can conduct heat and plastic can deform or deteriorate prematurely due to heat effects. As a daily donning routine, you should perform the following safety check:

Safety check

1. Inspect the uprights and joints for dents, scratches or cracks.
2. Straps, if present, should be secure and have no tears.
3. Make sure all rivets and screws are tight.
4. Check the locks, if present, to assure engagement.

If you have any problems, notice any undue wear, or observe any unsafe or unusual condition in the orthosis, contact your orthotist **immediately**.

Orthotist: _____

Phone: _____

Becker Orthopedic

635 Executive Drive, Troy, MI 48083-4576 USA
(248) 588-7480 • Fax (248) 588-2960
www.BeckerOrthopedic.com

EUAR: Acorn Regulatory Consultancy Service Limited

Knockmoris Cahir Co. Tipperary • Ireland • Postcode E21 R766
Phone 353 52 617 6706 • 353 52 614 8806

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