635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21

EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648

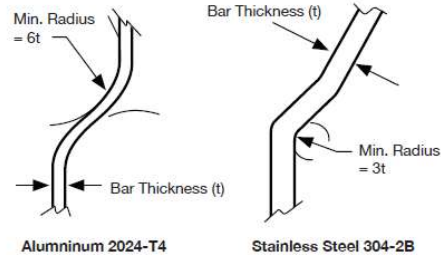
635 Executive Dr | Troy, MI 48083
P: 800-521-2192 | 248-588-7480
 BeckerOrthopedic.com | f t in

©2021 Becker Orthopedic Appliance Co.
 All rights reserved.
 Revision 05/01/21


EC REP

Acorn Regulatory Consultancy
 Services Limited
 Knockmorris Cahir Co. Tipperary Ireland,
 Postcode: E21 R766
 P 012 4626 8456
 F 012 4626 8648


NOT TO SCALE

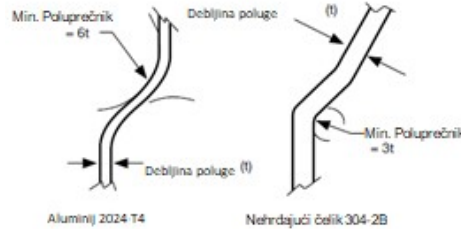An example for 1/4" Aluminum Uprights:
 Bend Radius is $6 \times 1/4" = 1-1/2"$. Draw a circle template with a $1-1/2"$ radius. Do not make a bend that exceeds the resulting arc.

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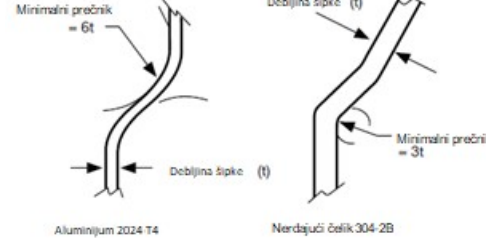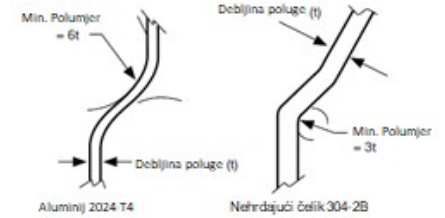
(Attach to Patient file at Delivery)

Congratulations - you have purchased the best. These joints are the strongest and most durable orthotic components. Additionally, Becker Orthopedic offers the largest selection of orthotic joints to enable you to precisely fit the needs of your patient.

This insert has two perforated sections with instructions for your patient and technician. Please remove the technical guidelines and give them to your technician to assure proper fabrication.

When you deliver the completed orthosis, please discuss basic maintenance procedures with your patient and give them the section entitled patient instructions.

This section should be signed by the patient as proof they have been given maintenance instructions and then stapled into your patient file.

Model #: _____ Lot #: _____

Patient Signature: _____

Date: _____

(Attach to Patient File at Delivery)

Patient Instructions (Give to Patient)

Your Certified Orthotist has selected the finest knee joints for your orthosis. It is important to keep the joints clean (wipe with a dry, lint free cloth) to assure smooth motion and safe engagement of the locks. For plastic or leather components use a mild detergent, rinse and dry thoroughly. When not in use your orthosis should be stored away from open flame or direct sunlight. Metal parts can conduct heat and plastic or leather parts could deform or deteriorate prematurely due to heat effects. The joints should also be lubricated periodically with the synthetic lubricant such as LPS. As a daily donning routine, you should perform the following safety check:

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Safety Check

1. Inspect uprights and knee joints for dents, scratches, or cracks.
 2. Straps should be secure and have no tears.
 3. Make sure all rivets and screws are tight.
 4. Check the locks to assure engagement.
- If you have any problems, notice any undue wear, or observe any unsafe or unusual condition in the orthosis, contact your Certified Orthotist **immediately**.

Orthotist or Firm: _____

Phone: () _____

(Please Give to Patient)

Technical Guidelines

(Give to Technician)

Please Follow these Guidelines:

1. Adjust distal bar for T.K.A. alignment
2. Do not modify anterior area of drop lock knee joints. Our design accommodates normal wear.
3. Do not exceed bending allowables of upright material. Minimum radius for 2024-T4 aluminum is six times bar thickness; 304 2B stainless steel 1/4 temper is three times thickness. Please see the reverse side for instructions
4. When drilling a hole in the uprights, be sure the hole is centered. Never "plug" unused holes in the upright.
5. Do not remove medial or lateral drop locks.
6. Remove surface scratches and dents on uprights to eliminate stress risers.
7. Lubricate joints with LPS or substitute before delivery.
8. Excessive tension on spring lever locks can result in premature wear of locking tips and joint heads.
9. This product is intended for use on a single patient.
10. Important: Apply high strength thread locking adhesive to all machine screws during final assembly of orthosis.

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